

Face Mask



1. Intended use:

Used for clinical medical staff in the process of invasive operation or noninvasive operation, covering the mouth, nose and chin of users, to protect patients from transfer of pathogens, microorganisms, body fluids, etc

2. Specification:

Specification	Size(mm)	Ear strap style	Type	sterile
LM-EFM-03	175x95	Ear loop	IIR	no
LM-EFM-04	175x95	Ear loop	II	no
LM-EFM-05	165x85	Ear loop	II	no
LM-EFM-06	175x95	Ear loop	I	no
LM-EFM-07	165x85	Ear loop	I	no

3. Components

The Face Mask consists of mask, nose clip and ear belt. The mask is composed of non-woven fabric and melt blown fabric.

4. Usage method

1. Take out the mask, white face inward, blue face outward;
2. Align the Face mask with the mouth and nose, with nose clip upward, and press the nose clip gently to make it fit with nose;
3. Hang the ear belt on the ear (or tie it to the head), and adjust the ear belt / tie to a comfortable state;
4. Adjust the wearing of the face mask to cover only the lower jaw of the nose, so as to obtain better protective effect.

Note: PP non-woven fabric and chinlon are used as the materials in contact with the body.

5. Performance

Bacterial filtration efficiency $\geq 98\%$ (type II, type IIR) or 95% (Type I);

Differential pressure < 60 Pa/cm² (type IIR) or 40 Pa/cm² (Type I, Type II).

The splash resistance pressure shall not be less than 16 kPa.(type IIR)

6. Contraindications

People who are allergic to non-woven fabric, polyester and spandex should use it with caution.

7. Warning

Use as soon as possible after opening.

It is recommended that the use time should not exceed 4 hours.

8. Cautions

This product is for single use only, DO NOT reuse it.

Use the product prior to the “Use By” date specified on the package.

Inspect all products prior to use. Do not use if the package is open or damaged.

Refer to the instructions for use before using this device.

Potential biological hazards exist after use. Please dispose products according to medical practice, local regulation and laws.

9. Useful life

2 years




10. Storage









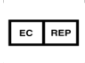





It shall be stored in a dry and cool place with good ventilation.

11. Transportation

The product shall be loaded and transported in covered carriage and cabin, and kept clean, avoid heavy pressure, sunshine and rain.

12. Symbols explanation

	Do not re-use		Manufacturer		Authorized representative in the European Community
---	---------------	---	--------------	---	---

	Do not use if package damaged		Date of manufacture		Caution
	Catalogue Number		Consult instructions for use		Keep away from sunlight
	Keep dry		Use-by date		Batch code
	Fragile, handle with		Sterilized using ethylene oxide		Latex free
	CE mark		Medical device		

13. Manufacture information


Lyncmed Medical Technology (Beijing) Co., Ltd.



Room 1601, Building No.2, Zhubang 2000 Business Building, Balizhuang Xili 99,
Chaoyang District, 100022 Beijing, PEOPLE'S REPUBLIC OF CHINA

+86 010 8646 8058

14. European representative

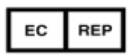

MedNet EC-Rep GmbH
Borkstrasse 10, 48163 Muenster, Germany

15. Other information

If you find a serious incident that has occurred in relation to the device, please reported to our company and the competent authority of the Member State in which the user and/or patient is established.